

BEURAY

PRODUCT LITERATURE | HSG AND SONOHYSTEROGRAPHY

Bellota Sonohysterography Catheter with Cervical Cone

Official Manufacturer IFU | English

The following pages reproduce the manufacturer's official document without alteration. For availability, quotations and support in Canada, please contact Beauray Inc.

CANADIAN SALES & SUPPORT


Beauray Inc.

info@beauray.co

+1 437 607 5116 | Thornhill, Ontario, Canada

For Canadian product inquiries, contact Beauray Inc.

Bellota Sonohysterography Catheter (with Cervical Cone)

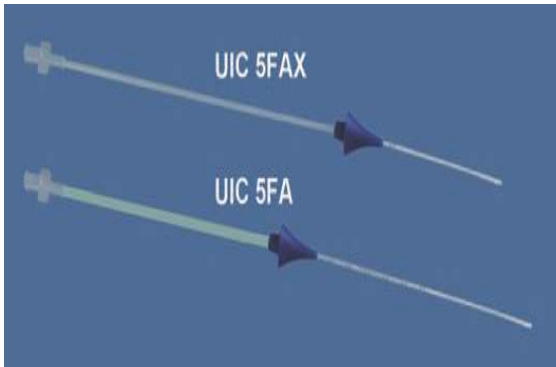

Bellota Sonohysterography Catheter (with Cervical Cone) is available in a 5 (f) size.

Not made with natural rubber latex

DESCRIPTION:

The catheter is to be used for sonohysterography procedures that aid in the analysis of uterine pathology. A placement sheath eliminates the need for a tenaculum. This catheter is designed for single patient use.

Product No.: UIC 5FA
UIC 5FAX

PACKAGING: 25 sterile catheters per box

INDICATIONS:

The Bellota Sonohysterography Catheter with Cervical Cone (Model: UIC 5FA) is used to access the uterine cavity for saline infusion Sonohysterography procedure.

CAUTION:

1. Sterile product unless package is opened or damaged.
2. Do not reuse for avoiding user may be infected by the microorganism.
3. This device is sold by or on the order of a physician.
4. If the sterile package is damaged or unintentionally opened before use, do not use.

CONTRAINDICATIONS:

The catheter type procedures are contraindicated in pregnancies, suspected pregnancies, active pelvic infections, or recent pelvic infections, severe uterine bleeding, gynecologic malignancies. This device should not be used for intrafallopian or intratubal procedure or for in-vitro fertilization.

PROCEDURE:

1. Remove the catheter from the sterile pouch.
2. Attach the saline source to the open luer and allow the saline to flow through the catheter removing air and flushing the saline channel.
3. Adjust the protective sheath so that just 1/2 to 1 cm of the catheter protrudes beyond the sheath.
4. Expose the cervix using a speculum and clean it with an antiseptic solution.
5. The single-lumen tube is featured with centimeter depth markings along its distal segment from 6-10cm to set the depth of insertion. It is helpful to place the catheter properly
6. Carefully advance the 5f catheter through the cervical canal and past the internal cervical OS, so that the catheter is within the uterine cavity while the catheter advanced and placing the acorn to avoid any fluid reflux.
7. Attach saline to open syringe and introduce through catheter slowly.
8. Remove the speculum and insert the transvaginal transducer into its proper position adjacent to the catheter.
9. Now activate the ultrasound system and inject saline as required by the procedure. Usually, 3cc to 20cc is sufficient saline for a proper diagnosis.
10. Perform Sonohysterography in the usual way.
11. Endometrial distention usually can be maintained throughout the procedure. If uterine distention is not visualized with ultrasound while saline is injected into the catheter or leaking is observed from the vagina, verify that the catheter is positioned correctly in the uterine cavity before continuing.
12. Upon completion of Sonohysterography, remove the catheter carefully from uterine cavity.

WARNINGS:

Dispose of medical waste properly in accordance with regulations after using.

STERILE EO

Sterile EO



Attention



Biological risks



Consult instructions for use



Warehouse temperature
10~40°C



Warehouse humidity
30~80%RH.



Not Made With
Natural Rubber Latex



Do not reuse



Batch code



Expiry date



Bioteque Taiwan Ltd.

7F., No. 431, 433, Baozhang Rd., Xizhi Dist., New Taipei City 22103, Taiwan