



PRODUCT LITERATURE | GYNECOLOGICAL INSTRUMENTS

Disposable Cervical Dilator - Instructions for Use

Official Manufacturer IFU | English

The following pages reproduce the manufacturer's official document without alteration. For availability, quotations and support in Canada, please contact Beauray Inc.

CANADIAN SALES & SUPPORT

Beauray Inc.

info@beauray.co

+1 437 607 5116 | Thornhill, Ontario, Canada

For Canadian product inquiries, contact Beauray Inc.

Cervical Dilator



Not made with natural rubber latex

DESCRIPTION:

Disposable Cervical Dilators are designed for single-patient use only after sterilizing. They are made of Polyethylene.

Product No.:

OS265BA-1/OS265BA-2 / OS265BA-3

CAUTION:

1. Federal law restricts this device to sale by or on the order of a physician.
2. Do not reuse for avoiding user may be infected by the microorganism.
3. If the sterile package is damaged or unintentionally opened before use, do not use.
4. Do not use the device after expiration date.

PRECAUTION:

Check the packaging for damage for this may compromise sterility. Also, visually inspect the dilators for any surface damage.

INDICATIONS:

These Cervical Dilators are especially useful in locating and dilating the external OS, the cervical canal and the internal OS. They are particularly valuable instruments to be used on post-menopausal patients with cervical stenosis.

CONTRAINDICATIONS:

Cervical dilators should not be used under the following conditions:

1. Acute genital track or pelvic infections.
2. A pregnancy or the suspicion of a pregnancy.
3. Any invasive cancer that is visible upon examination.
4. Any non-compliant patient.

Disposable Cervical Dilators is smooth and flexible, light in weight and can be easily inserted and removed for their intended applications. Please be aware that excessive force should be avoided and these instruments should not be used for prolonged periods of time.

Each dilator is manufactured to exacting standards. They are flexible very smooth delivering excellent patient comfort for their intended indication.

The OS Finder

A single molded band above the tactile handle and opposite the distal end of the device identifies the disposable OS Finder. The OS Finder may be used to find and gently dilate the external OS. Gradual dilation in combination with its tip flexibility and smooth surface will ensure maximum patient comfort. The sound of the OS Finder is apered3mm to an intermediate diameter of 3.8mm.

The Canal Locator

Two molded bands above the tactile handle identify the Canal Locator from the OS Finder and the Fundus Locator. Its sound or longer tapered tip is 45mm and extends to an intermediate diameter of 3.8mm. The Canal Locator may be used to slowly dilate the length of the cervical canal. Its flexible and smooth surface will provide maximum patient comfort.

The Fundus Dilator


The Fundus Dilator has three molded bands opposite the distal tip that identifies this instrument from the others in the 3 piece OS Finder Set. The sound or tapered length from the distal tip is 85mm. The smooth and flexible Fundus Dilator is designed to pass through the entire length of the cervix to dilate the internal OS.

WARNINGS:

1. This device should be used with caution to avoid uterine perforation.
2. Dispose of medical waste properly in accordance with regulations after using.



Warehouse temperature 10~40°C

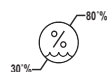

Warehouse humidity 30~80%RH.



Not Made With Natural Rubber Latex



Do not reuse



Batch code



Expiry date



Attention



Biological risks



Consult instructions for use



Sterile EO



Bioteque Taiwan Ltd.

7F., No. 431, 433, Baozhang Rd., Xizhi Dist., New Taipei City 22103, Taiwan