

BEURAY

PRODUCT LITERATURE | HSG AND SONOHYSTEROGRAPHY

H/S Catheter Stylet - Instructions for Use

Official Manufacturer IFU | English

The following pages reproduce the manufacturer's official document without alteration. For availability, quotations and support in Canada, please contact Beauray Inc.

CANADIAN SALES & SUPPORT


Beauray Inc.

info@beauray.co

+1 437 607 5116 | Thornhill, Ontario, Canada

For Canadian product inquiries, contact Beauray Inc.

H/S Catheter Stylet

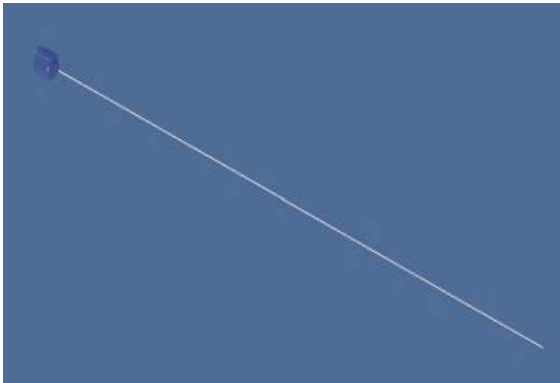

Not made with natural rubber latex

DESCRIPTION:

The H/S Catheter Stylet is designed to use to gain ureteral access, to establish a tract, and to assist in the placement during gynecologic procedures.

Product Code: BA001

PRECAUTIONS:

1. Use caution not to force or over manipulate the H/S Catheter Stylet when gaining access.
2. When using H/S Catheter Stylet through a metal cannula, use caution as damage may occur to outer.
3. The H/S Catheter Stylet is not intended for PTCA use.

INSTRUCTIONS:

1. Using aseptic technique, remove the H/S Catheter Stylet from its outer packaging and place in the sterile field.
2. Remove the H/S Catheter Stylet from the holder.
3. Adjust mandril position by advancing or withdrawing the mandril within the H/S Catheter Stylet to achieve the required degree of flexibility when using the H/S Catheter Stylet.
4. Advance the flexible end of the H/S Catheter Stylet into the HSG catheter and position to the desired location.

Note: The size and length of the device must be taken into consideration to ensure proper fit between H/S Catheter Stylet and device.


5. After H/S Catheter Stylet use complete, removed from patient and discard.

CAUTION:

1. Sterile product unless package is opened or damaged.
2. Do not reuse for avoiding user may be infected by the microorganism.
3. This device is sold by or on the order of a physician.
4. If the sterile package is damaged or unintentionally opened before use, do not use.

WARNINGS:

Dispose of medical waste properly in accordance with regulations after using.

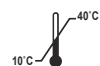

Warehouse temperature 10~40°C



Warehouse humidity 30~80%RH.



Not Made With Natural Rubber Latex



Do not reuse



Batch code



Expiry date



Attention



Biological risks



Consult instructions for use



Sterile EO



Bioteque Taiwan Ltd.

7F., No. 431, 433, Baozhang Rd., Xizhi Dist., New Taipei City 22103, Taiwan