



BEAURAY

PRODUCT LITERATURE | PESSARIES

Use and Care of Pessaries

Official Manufacturer Patient guide | English

The following pages reproduce the manufacturer's official document without alteration. For availability, quotations and support in Canada, please contact Beauray Inc.

CANADIAN SALES & SUPPORT

Beauray Inc.

info@beauray.co

+1 437 607 5116 | Thornhill, Ontario, Canada

For Canadian product inquiries, contact Beauray Inc.

The vaginal pessary has a long history, dating back thousands of years! Ancient pessaries were stones, pomegranates, linen, wool, and sponges. In the middle ages brass, cork, gold, and silver were the new materials. During the 1800's, the ring pessary was developed. Vaginal pessaries were refined in the 1900's with advances in material science. Now there are silicone ones in many shapes and sizes. Few medical inventions have stood the test of time so well as the pessary.

*Compliments of Bioteque America
Written by: Margaret G. Dorsey*

Common Uses of Pessaries

To Improve or restore continence.



For diagnosis to determine if surgery is necessary.



To delay or preclude surgery.



As an alternative to surgery.



To manage uterine prolapse and vaginal wall hernias.



To relieve a cystocele (bladder prolapse) or a rectocele.



To relieve lower back pain caused by retroversion.



Diagnosis and therapy in obstetrics.



To prevent miscarriage by relieving pressure on a weak cervix.



The Use And Care Of Pessaries



If you are a candidate for a pessary, you will find the following information useful.

A pessary is a vaginal prosthesis that has evolved over centuries to become an effective medical device to relieve the stress of a cystocele, a rectocele, a uterine or bladder prolapse, or the problems associated with urinary incontinence. Pessaries come in various, shapes, sizes, and materials. They are perfectly safe and comfortable for long term usage.

Commonly asked questions:

Will I always have to use this pessary?

Not necessarily. Although pessaries are an alternative to surgery, your vaginal muscles may strengthen to the extent that your pessary is no longer required. Sometimes pessaries are only required for strenuous exercises like jogging or hiking. Certainly, if you are a candidate for surgery, your pessary is only temporary until corrective surgery can be performed.

Can a pessary hurt or become uncomfortable?

A properly fitted pessary will never cause pain or discomfort. You should be able to perform your normal daily activities without any difficulty.

How often should I remove my pessary?

It is recommended that you remove your pessary daily to clean it using mild soap and water. However, if you have difficulty removing your pessary, contact your

healthcare provider and make an appointment for removal and cleaning. Do not be alarmed if your appointment date is weeks away. It is not unusual for pessaries to remain in place from 4 to 12 weeks.

What is my responsibility?

Your responsibility is to follow your healthcare provider's instructions, wear your pessary, and keep all follow-up appointments. If you should move, be sure to notify your healthcare provider.

Will my pessary set off the alarm at the airport?

Absolutely not.

Can I have sex with my pessary in place?

You certainly can. If you need to remove your pessary, then do so.

Will my pessary prevent pregnancy?

No, it will not. A pessary is not a contraceptive device. If you are sexually active, wear a pessary and need to avoid pregnancy, then discuss this with your healthcare provider.

Can my pessary fall out?

Yes it can. If your pessary is too small or you need a different type of pessary, sometimes it will fall out. Whenever this happens, simply clean your pessary with mild soap and water and reinsert it. If it is a persistent problem, then call your healthcare provider for an appointment. You may need a different size or type.

Daily Care and Helpful Hints:

I can't remove my pessary, now what?

If you want to remove your pessary but can't, don't panic. Simply call your healthcare provider and schedule an appointment.

My pessary seems to have moved out of position and is uncomfortable.

Wash your hands, lie down and push the pessary up as far as you can. Don't worry, it can't get lost. It's impossible to push your pessary into your abdomen.

I see some discharge and bleeding. What should I do?

Sometimes pessaries can irritate body tissue causing a slight discharge or bleeding. Call your healthcare provider if it persists.

What do I do about daily bathing?

Do not change your normal bathing routine because you are wearing a pessary.

Should I douche?

You do not have to douche unless instructed to do so by your healthcare provider.

Bioteque
America Inc.

1-800-889-9008
Fax: 408-526-9399

visit our website at: www.bioteque.com
or e-mail us at: sales@bioteque.com