

BEAURAY

PRODUCT LITERATURE | GYNECOLOGICAL INSTRUMENTS

Uterine Sound - Instructions for Use

Official Manufacturer IFU | English

The following pages reproduce the manufacturer's official document without alteration. For availability, quotations and support in Canada, please contact Beauray Inc.

CANADIAN SALES & SUPPORT


Beauray Inc.

info@beauray.co

+1 437 607 5116 | Thornhill, Ontario, Canada

For Canadian product inquiries, contact Beauray Inc.

Uterine Sound

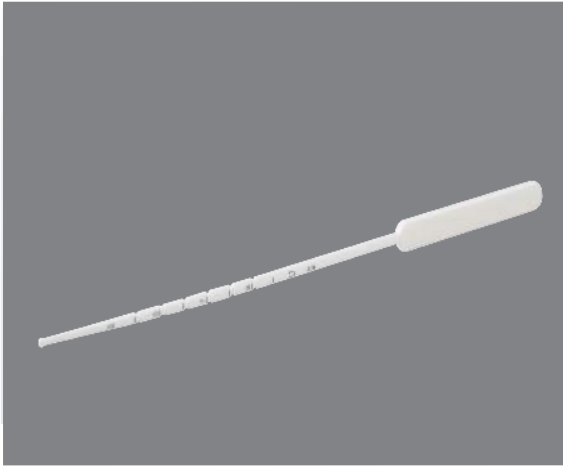

Not made with natural rubber latex

DESCRIPTION:

The Uterine Sound is used to gauge the depth and position of the uterine cavity. It is also helpful with the placement of IUDs and the verification of the intrauterine position and length for HSG procedures. The proximal portion of the sound is 3.5 mm in diameter tapering to 2.5 mm at the distal portion with a bulbous tip to reduce the risk of perforation. Malleable to adapt to anatomical variations. It is a sterile, disposable, single-use only device.

Product No.:

BA100

CAUTION:

1. Sterile product unless package is opened or damaged.
2. Do not reuse for avoiding user may be infected by the microorganism.
3. This device is sold by or on the order of a physician.
4. If the sterile package is damaged or unintentionally opened before use, do not use.

PRECAUTIONS:


1. Do not use if package seal is damaged or broken.
2. Minor bleeding, cervical inflammation, and abdominal cramps may occur.

DIRECTIONS FOR USE:

The Uterine Sound is a single-use sound. It is made of white, malleable, plastic material. Prior to use, the sound can be curved between the thumb and finger in order to follow the curve of the cervical canal. The graduations are in centimeters and allow for the measurement of the uterine cavity. Dispose of the Uterine Sound in accordance with all applicable Federal, State and local Medical / Hazardous waste practices.

WARNINGS:

1. Gently introduce and manipulate the tip of the sound through the cervical canal. Forced entry could create a false tunnel and bleeding.
2. Dispose of medical waste properly in accordance with regulations after using.

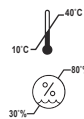

Warehouse temperature 10~40°C

Warehouse humidity 30~80%RH.



Not Made With Natural Rubber Latex



Attention



Expiry date



Biological risks



Consult instructions for use



Do not reuse



Batch code



Sterile EO



Bioteque Taiwan Ltd.

7F., No. 431, 433, Baozhang Rd., Xizhi Dist., New Taipei City 22103, Taiwan